

Connecting your iTero scanner

How to send cases to Bridge Dental Lab from your iTero scanner.

iTero scanners (Align Technology) require iTero support to add a new lab to your account. This is a one-time setup — once we're added to your favourites, sending future cases takes a single tap.

LAB CONNECTION DETAILS

Bridge Dental Lab · iTero Lab ID: **54927** · Registered email: **scan@bridgedentallab.com**

Step-by-step

1. On your iTero scanner, tap **Find a Laboratory**, or visit itero.com/en-gb and use the lab locator.
2. If Bridge Dental Lab doesn't appear, contact iTero support and ask them to add us using ID **54927** or email **scan@bridgedentallab.com**.
3. Restart the scanner once we've been added.
4. Select **Bridge Dental Lab** from your favourites when sending cases.
5. Complete your scan (prep, opposing, bite), fill in the digital Rx in MyiTero, then tap **Send**.

Useful links

- iTero UK support: itero.com/en-gb/support
- MyiTero portal: my.itero.com

What to include with every digital case

- **Appliance required**
- **Both arches scanned**
- **Bites** if required
- **Which practice** the case is for, if you are a multi-practice dentist

Please send any gold service scans before 11am so we can ensure your appliance will be back with you the following day.

Need help connecting your scanner?

Phone: 01202 486113

WhatsApp: 07307 858772

Email: scan@bridgedentallab.com

Web: bridgedentallab.co.uk

We accept scans from all major scanners with no extra fees — free 3D printed models with every appliance order.

Bridge Dental Laboratory Ltd

Unit 10 & 11, Sea Vixen Industrial Estate
Wilverley Road, Christchurch, Dorset BH23 3RU

01202 486113 · scan@bridgedentallab.com
bridgedentallab.co.uk

DAMAS-accredited · GDC-registered technicians